## CONNECT WITH US!

[facebook.com/cambridgemaskco](https://facebook.com/cambridgemaskco)

[cambridgemask](https://cambridgemask)

[twitter.com/cambridgemaskco](https://twitter.com/cambridgemaskco)

[info@cambridgemask.com](mailto:info@cambridgemask.com)

[WWW.CAMBRIDGEMASK.COM](http://WWW.CAMBRIDGEMASK.COM)

## USER MANUAL

# CAMBRIDGE MASK CO

[WWW.CAMBRIDGEMASK.COM](http://WWW.CAMBRIDGEMASK.COM)

**CE 0493** EN149:2001 + A1:2009 FFP2 R D

Cambridge Environmental Solutions Ltd. RM 1001-2 10/F Wilson House, 19-27 Wyndham Street  
Central Hong Kong. EU Importer Cambridge Mask Company, Unit 179, 23 King Street,  
Cambridge, CB1 1AH, UK Conformity assessment to EU PPE Regulation, (EU) 2016/425,  
carried out by Centexbel, Technologiepark 70, 9052 Gent-Zwijnaarde, Belgium.

Dear Customer,

Thank you for choosing Cambridge Mask. We work hard to make sure you and those you care about can get outside and enjoy life without worrying about the air you breathe or the way you look.

If you have any feedback about our masks, please get in touch via [info@cambridgemask.com](mailto:info@cambridgemask.com).

*Christopher J Dobbins*

## CONTENTS

English	2
Spanish	6
German	10
French	14
Chinese	18
Korean	22
Japanese	26
Hindi	30
Arabic	34
Bosnia and Herzegovina	38

## YOUR CAMBRIDGE MASK


Thank you for choosing Cambridge Mask. We are proud of the high quality product that we make here in the UK. We love to hear feedback. If you have questions or comments about how we could improve our service or products, please contact us at [info@cambridgemask.com](mailto:info@cambridgemask.com).

### CUSTOM BRAND MASKS

Did you know we can custom brand masks?

Cambridge Mask Co can create custom branded masks for your organisation, perfect for staff, valued customers or potential clients.

Orders can be as small as 200 units. For more information visit [www.cambridgemask.com](http://www.cambridgemask.com)

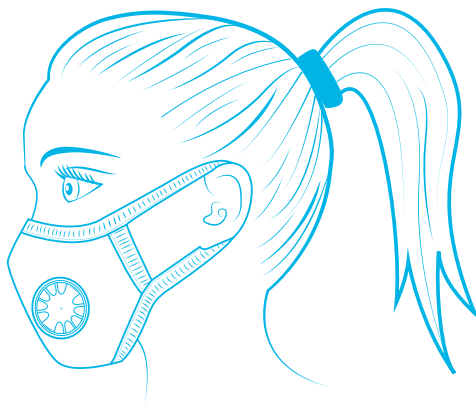### WARNING

Mask must be fitted properly to filter effectively  
 Infants and children should be supervised when wearing mask  
 Mask does not filter industrial chemicals or oil-based particles such as paint  
 Do not exceed recommended maximum hours of usage based on the usage instructions

## USAGE GUIDLINES

Pollution Level	Air Quality Conditions (AQI)	Description	Replace After
Good	Below 50	Clear visibility to horizon	Mask Not needed
Moderate/Unhealthy for Sensitive Groups	51-150	Cannot see pollution with the naked eye, slight haze on the horizon	340 hours
Unhealthy/Very Unhealthy	151-300	Haze at 2-3 km visibility	220 hours
Hazardous	301-500	Clouds of pollution, visibility around 1km.	180 hours
Beyond Index	Over 500	Visibility below 600 meters. Fog-like clouds of pollution clearly visible even in bright sunlight.	90 hours

Note: Bandings in this table are based on the US EPA Standard. Some stratification has been collated for ease of use.

### SIZING

**XL**  
175x220mm  
90kg

**L**  
170x200mm  
65-89kg

**M**  
150x190mm  
33-64kg

**S**  
140x170mm  
11-32kg


**XS:**  
120x135mm  
6-10kg

Sizing of the masks is based on the weight of the person wearing it.


## ABOUT YOUR MASK

This non-disposable half face respirator can be worn far longer than most disposable industrial masks. However once the filters are “full” of pollutants, the mask needs to be changed. Dirt can be washed off the mask but the tiny pollution particles are permanently bonded to the filter. It is not possible to wash them out. Replace your mask once you feel the breathing resistance increasing significantly from your normal use.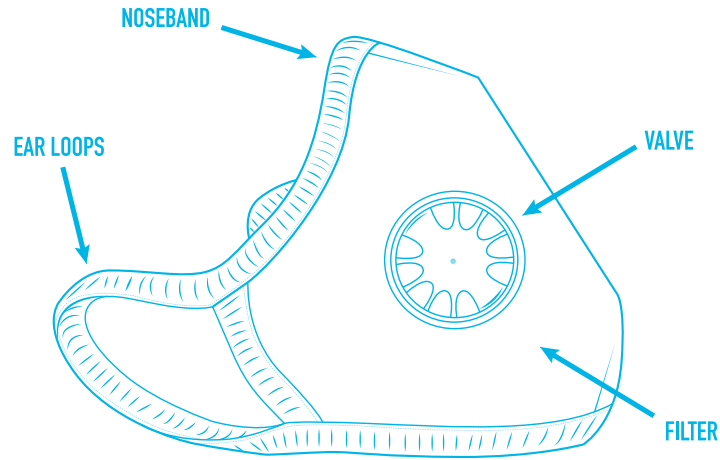


## FITTING YOUR MASK


**1** The ear straps are designed to fit behind the ears to make the mask comfortable to wear. Please make sure there is a tight seal. Press the mask to your face when you put it on to help form a seal.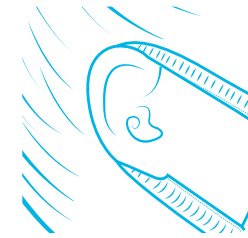

**2** When you wear the mask be sure to squeeze the noseband so that there is a tight seal around all of the face. This ensures that the air is flowing through the mask and not around it.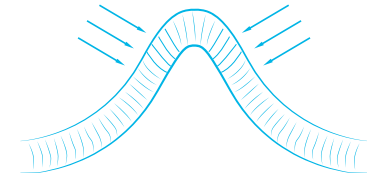

**3** While the whole mask is the filter, the valve is there to help reduce humidity in hot climates or while cycling / exercising.

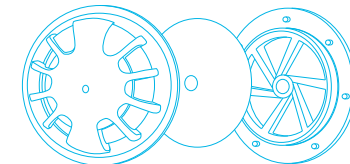Front Valve Case | Silicone Membrane | Back Valve Case

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