PLEASE PRINT THE WARRANTY ON THE BACK SIDE OF THE INSTRUCTION SHEET.

CP2965-XXX-01

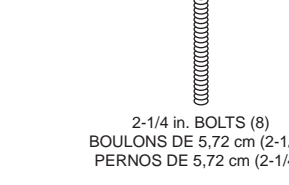
Tub Transfer Bench
Banc de transfert
Banco para transferirse a la tina

TOOLS REQUIRED:

Phillips Screwdriver
Wrench (provided)

Before beginning assembly, identify all parts.

- Backrest (1)
- Seat slats (3)
- Side braces (2)
- Backrest supports (2)
- Support arm (1)
- Adjustable legs (2)
- 2-1/4 in. bolts (8)
- 1/4 in. lock nuts (8)
- 2 in. screws (4)
- 2-1/4 in. screws (4)
- Cap nuts (4)
- 1-1/4 in. screws (2)
- 5/8 in. sheet metal screws (2)
- Wrench (1)
- Curved washers (8)

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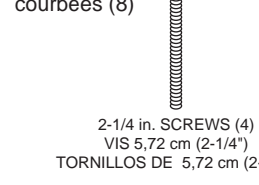
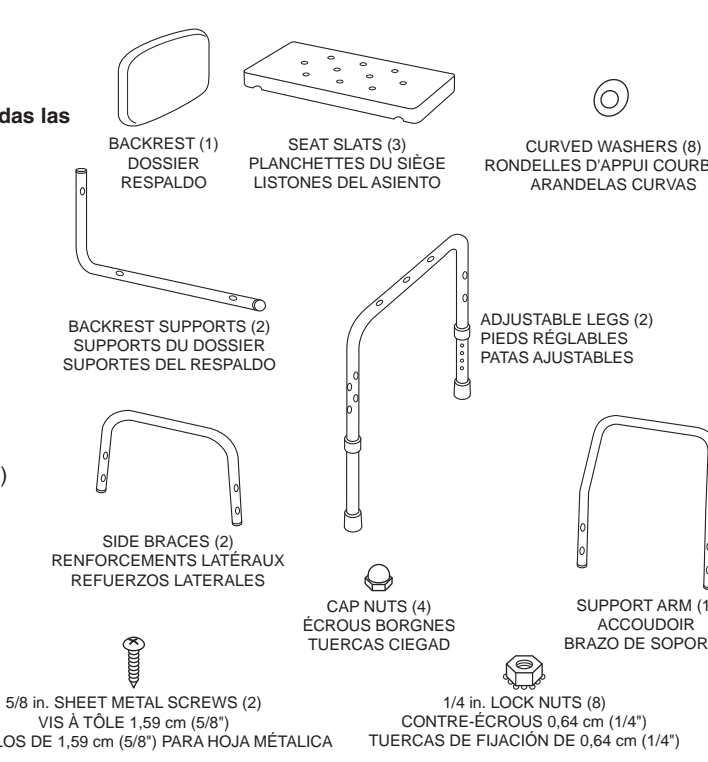
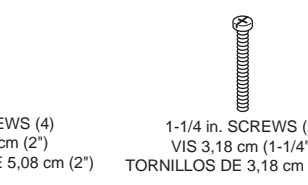
ASSEMBLY INSTRUCTIONS:

STEP 1. FRAME ASSEMBLY (SEE FIG. 1)

1. Place one adjustable leg in upright position with foot pieces downward.
2. Place side brace (with tube ends facing down) to the side of leg and align screw holes with holes in leg.
3. Insert curved washer onto 2-1/4 in. bolt, then insert through leg holes into side brace, and fasten securely with 1/4 in. lock nuts using the wrench provided.
4. For the second side brace, repeat steps 1 to 3. Make sure support arm holes face the same end.

CAUTION: Be careful not to over-tighten bench frame assembly bolts and nuts. Over-tightening can cause damage to tubing.

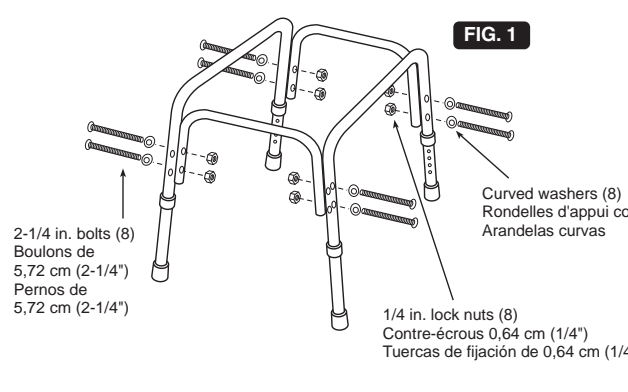
STEP 2. SUPPORT ARM ASSEMBLY (SEE FIG. 2)

1. Place support arm to the side of bench frame with holes, making sure that arm angles away from frame.
2. Insert four 2 in. screws through support arm into bench frame and fasten securely with four cap nuts using the wrench provided.

STEP 3. BACKREST SUPPORT ASSEMBLY (SEE FIG. 3)

The backrest can be assembled facing either left or right as you look at the side of the bench. Look at the bathtub you will be using. The backrest should be on the side of the bench that sits in the tub and should face the tub spout.

1. Assemble backrest support tubes to the plastic backrest by sliding the shorter tube sides with only one hole into receiving holes of backrest.
2. Secure backrest to support tubes using two 5/8 in. sheet metal screws.
3. Assemble backrest tubes to bench frame using four 2-1/4 in. screws secured through bench frame tubing to the bottom of two plastic seat slats.
4. Assemble the remaining plastic seat slat to the bench frame using two 1-1/4 in. screws and securely fasten.

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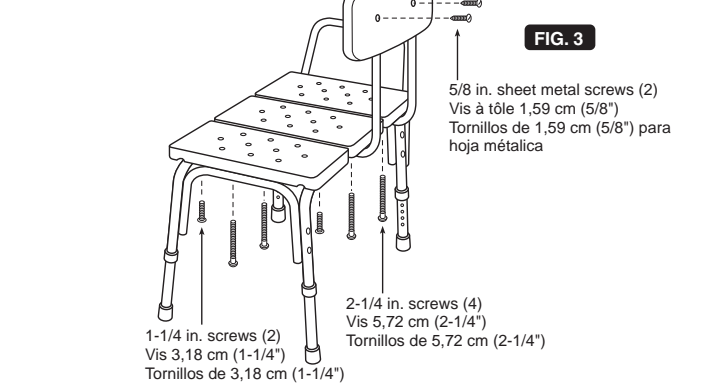
USER INSTRUCTIONS:

Place transfer bench in bathtub facing tub spout, with two legs sitting inside the tub and two legs outside the tub. Adjust leg height by depressing detent buttons on legs and sliding legs (until button pops into place) so that bench is at a level height while two legs are inside the tub and two legs are outside the tub.

When entering tub: Sit on side of bench that is outside of tub. Carefully slide over to center of bench, while lifting each leg over the tub wall. It is recommended that you consult a health care professional for instructions specific to your needs.

WARNING: Before use of the bench, check that all components are fastened securely and that buttons on adjustable legs are popped out completely and fully engaged in selected holes. If components are damaged or missing, contact your dealer immediately. **DO NOT USE SUBSTITUTE PARTS.** Do not jump or stand on bench. This product has a maximum user weight of **250 pounds**. Liberty Hardware Manufacturing Corporation assumes no liability for any damage or injury caused by improper installation, assembly or use of this product.

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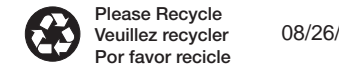
©2009 Liberty Hardware Manufacturing Corporation®, A MASCO COMPANY

08/26/09

SAFETY FIRST LIMITED 10 YEAR WARRANTY

TUB TRANSFER BENCH
CP2965-XXX-01 INSTRUCTION/WARRANTY SHEET
11" x 17" TWO-SIDED