

Instructions For Use

Inserting Your Earplugs

Insert the plug by grasping the end cap and inserting the eartip until the largest flange is sealed in the ear canal entrance. To ease insertion, moisten the plug and pull the ear outward and upward. Earplugs must be inserted correctly for optimal effectiveness.

Removing Your Earplugs

Remove with a slow twisting motion.

Cleaning

- Clean after each use.
- Wipe clean with a damp cloth as needed.
- Do not immerse earplugs in water.
- Do not clean with harsh chemicals.
- Alcohol is not recommended.

Replacement

ER•20XS are reusable, but with repeated use, the eartips may lose elasticity. For best performance and retention, if you use your ER•20XS constantly, consider replacing them every 3-6 months.

