

Foot Stool WITH HANDRAIL

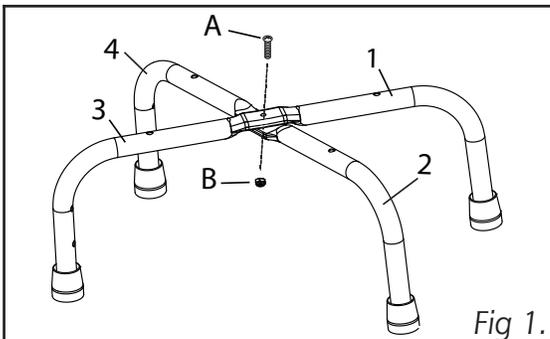


Fig 1.

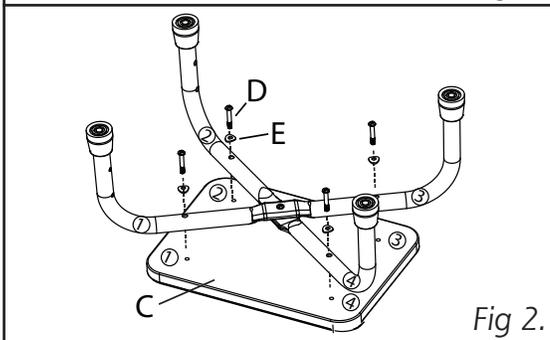


Fig 2.

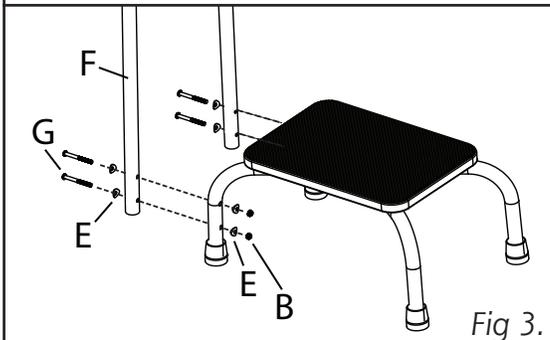


Fig 3.

Weight Capacity: 250 lbs.

Model: 539-1902-0099

Assembly Instructions

- ▶ 1. In the upright position, crisscross the leg assemblies so the center flattened areas are flush. The legs labeled (1) and (3) should be on top. Be sure the legs numbered 3 and 4 are on the same side. Insert the 7/8" Base Screw (A) through both leg assemblies and secure with a Nut (B) as shown in Fig. 1. Using a Philips head screwdriver, tighten the screw.
DO NOT OVER TIGHTEN AS IT COULD STRIP THE THREADS.
- ▶ 2. Place the Footrest Base (C) upside down on a flat surface with the screw holes facing up. With the leg assembly upside down, align the holes in the legs with the threaded sockets in the footrest next to the corresponding numbers as shown in FIG. 2.
Insert the 1-7/8" Leg Assembly Screw (D) through a Washer (E) and into each of the leg assemblies. Using a Philips head screwdriver, loosely tighten each of the screws.
- ▶ 3. Attach the Handrail (F) by aligning the four holes in the handrail with the four holes on legs 3 and 4. Insert the 2-1/4" Handrail Screw (G) through a Washer (E) into leg assembly. On the back side, add another Washer (E) and secure with a Nut (B). Do this for the remaining the three holes.
- ▶ 4. Go back and tighten all screws and nuts.
DO NOT OVER TIGHTEN AS IT COULD STRIP THE THREADS.

Components List

- ▶ A - Base Screw 7/8" (1)
- ▶ B - Nut (5)
- ▶ C - Footrest Base
- ▶ D - Leg Assembly Screw 1-7/8" (4)
- ▶ E - Washers (8)
- ▶ F - Handrail
- ▶ G - Handrail Screw 2-1/4" (4)

WARNING

This product contains natural rubber latex which may cause allergic reactions.

Manufactured for:
Briggs Medical Service Company
1931 Norman Drive South
Waukegan, IL 60085 USA
Made in China
#910-1902-0099 8/10
©2010 Briggs Medical Service Company

If any parts are missing, do not return to the place of purchase.
Contact us at 800-622-4714.

Safety

Security

Durability