


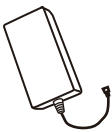


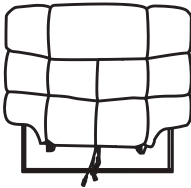
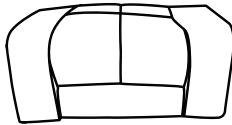





Assembly Instruction

Lift Chair


Your recliner comes packaged in one box.
Carefully remove all of the components from
the packing and set side for assembly.

Package Contents

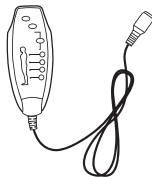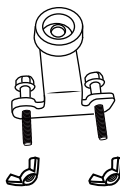			<div>CAUTION:</div> <div>DO NOT use any sharp objects to open plastic wrapped components as damage to product or components may results.</div>	
A. BACK	B. SEAT	C.2*ARMREST		
				
D. TRANSFORMER	E.PLUG	F . POWER CORD	G. MASSAGE REMOTE	H. 2*ANTI-FALL BRACKETS

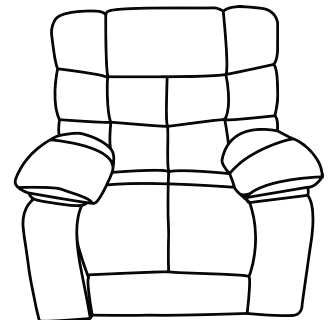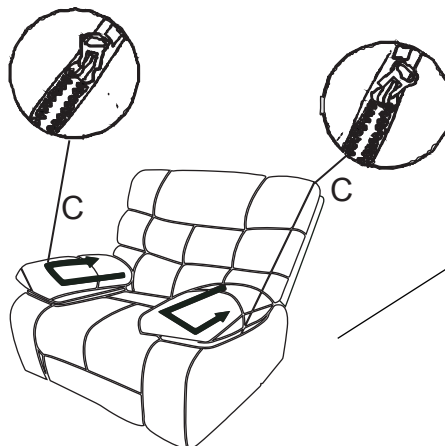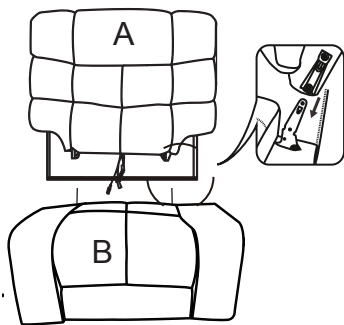


STEP1

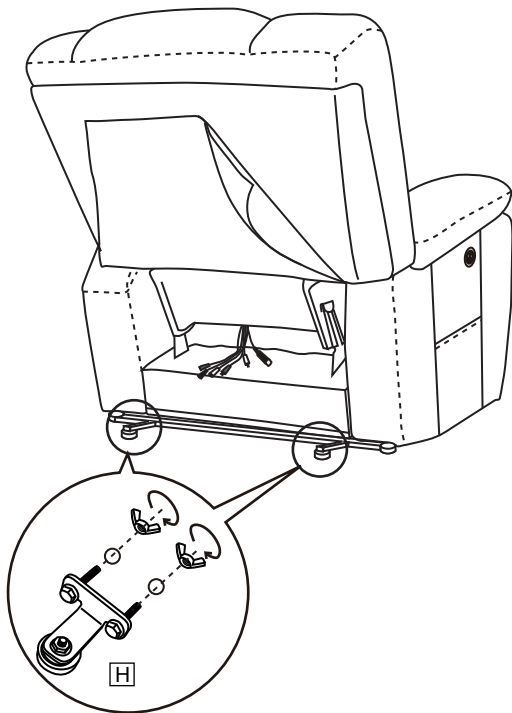


MUST BE ASSEMBLED

STEP2


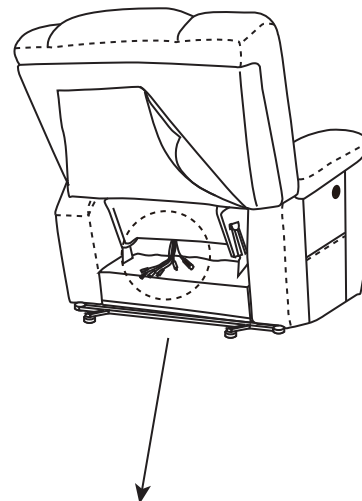


NOTE : Two same color wires connect

No.1 and No.2 wire connect with the backrest and chair base.
No.3 wire connects with backrest and power cord(F).
No.4 wire connects with backrest and armrest..

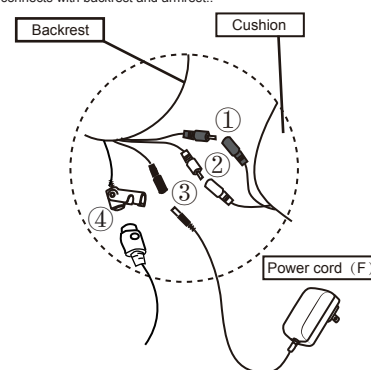


STEP3

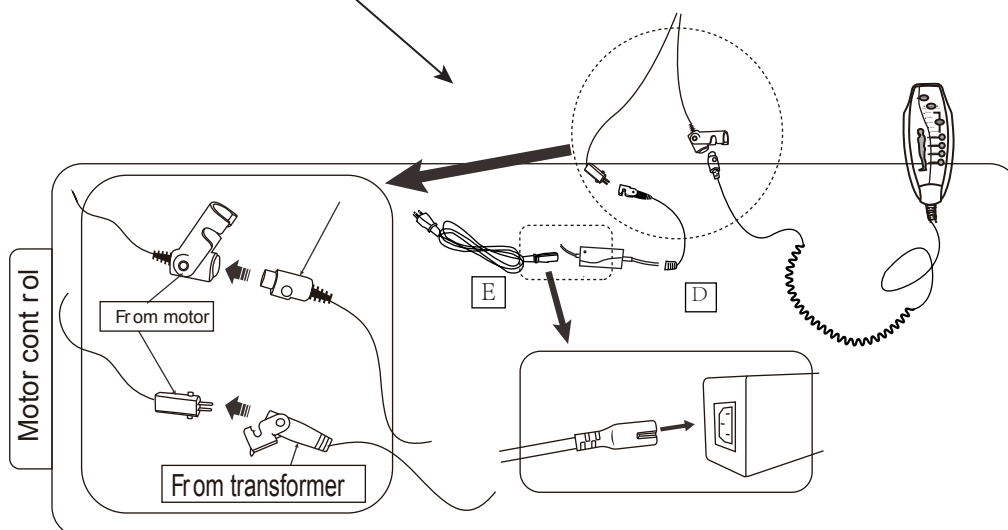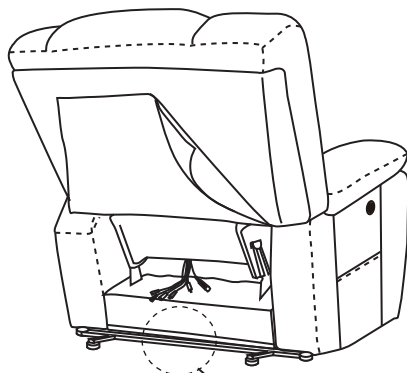


Remote Operation Instruction

How to recline?

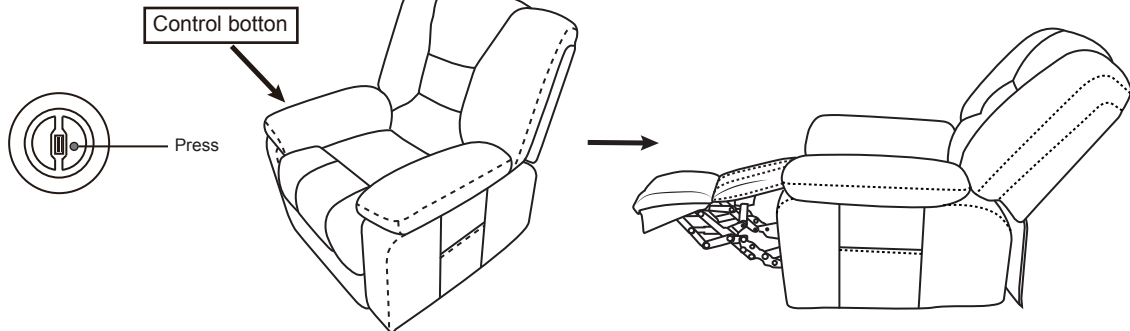

How to get footrest latch down and stand?

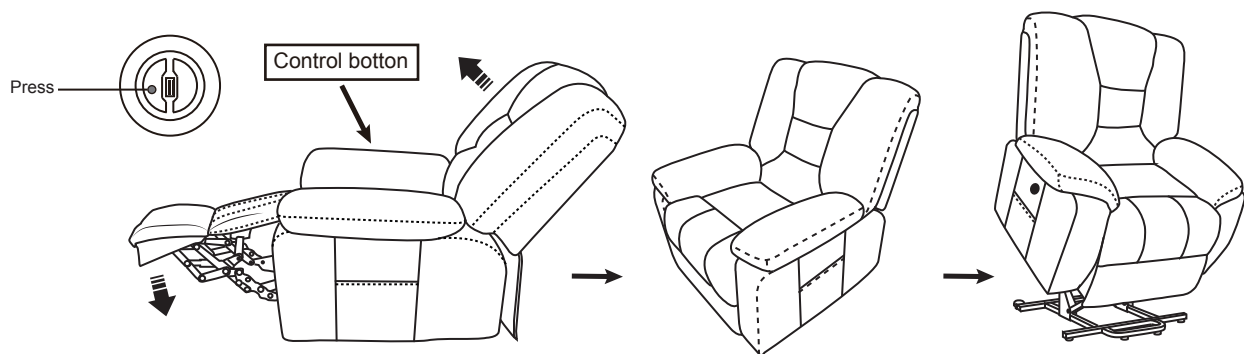

Warnings

1.Q: Missing parts when received.

A: Please kindly send us messages to let us know the name of missing part regarding to the first page of instructions. We will send replacement to you asap.

2.Q: Some parts damaged when arrived.


A: Please send clear photos about damaged part to us as attachment, we will check and offer you best solution soon.

3. Q: The chair is stuck on lifting or reclining position, and no response when you press the lifting or reclining button.

A: Please kindly check if the transformer (D) and control panel are connected well with lift motor. Sometimes the transformer(D) and control panel may fall off from the motor. If they connect well, then check the lift motor if it has some anomaly, like some noise comes out when and control panel may fall off from the motor is defective, please contact us for replacement.

Assembly Instruction


Disconnect seating and back

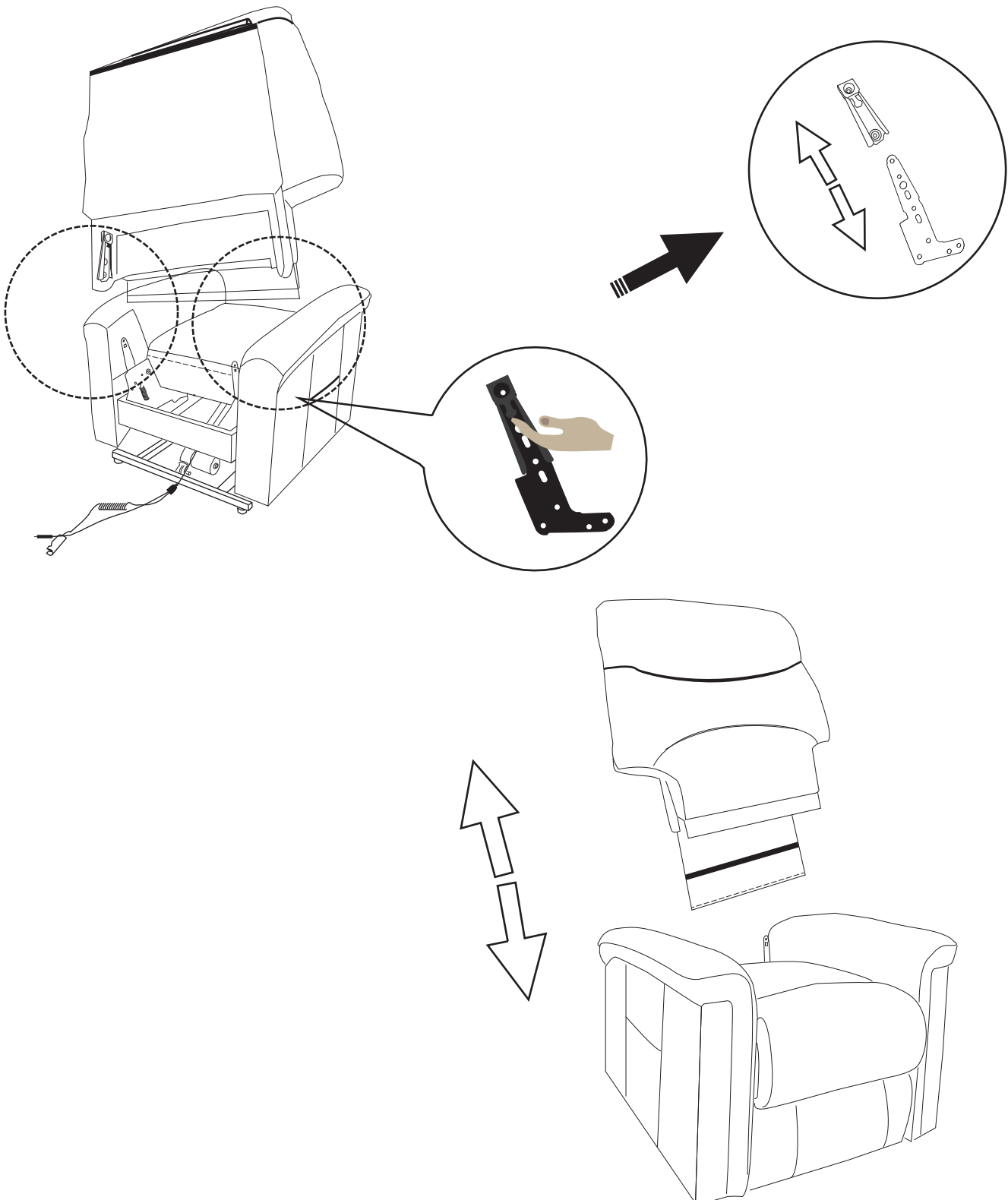


Assembly Instruction

Different Positions


RECLINING

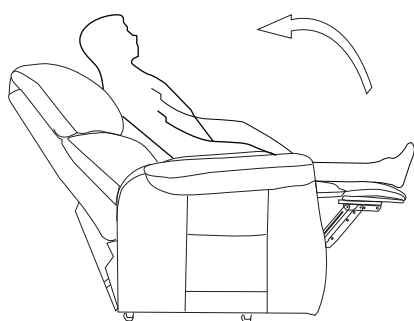

SITTING

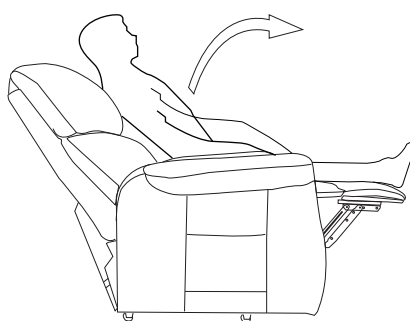

LIFTING

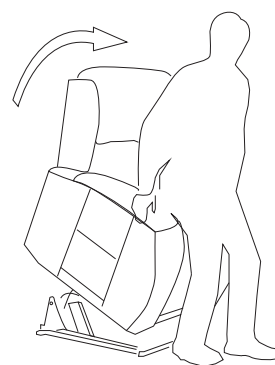

15 inches
make
enough space
for reclining back

