
# Telescoping **ADJUSTABLE WHEELCHAIR RAMP**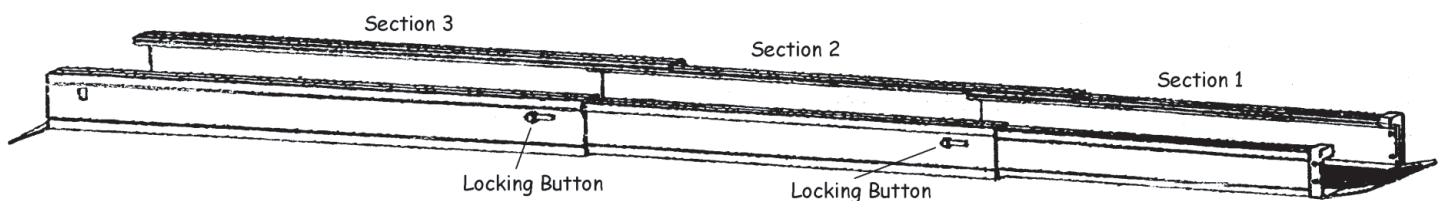

## Assembly Instructions:

The ramps are packed as three together collapsed into one. To extend the ramps, pull Sections 1 and 2 until the locking buttons are fully extended into the holes.

**CAUTION:** Be sure the locking buttons are fully engaged, otherwise the ramp could become unstable and cause injury.

## User Instructions:

A 9.5 degree grade is acceptable for portable ramps used by occupied wheelchairs with the assistance of another person.

To determine the length of ramp, first determine your rise. The rise is the vertical measurement between the ground and where the top of the ramp is going to be placed.

Next take the total amount of rise and divide by 2. For example: a 10" rise divided by 2 would require 5' of ramp.

**NOTE:** If the ground is not level, consideration must be taken for the variance in slope.

Distributed by:  
Duro-Med Industries  
Div. of DMS Holdings Inc.  
1931 Norman Drive South  
Waukegan, IL 60085 USA  
www.duromed.com

Made in Taiwan R.O.C.

Toll-Free Customer  
Care Help Line  
877-505-6190  
#91-032-4094 02/07  
©2007 Duro-Med Industries

### **NOTE:**

Weight Capacity: 660 lbs.  
Maximum Length: 5 ft. fully extended  
Width: 7"

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